## EN Instructions for use

### Purpose PSA (Regulation 2016/425)

BlueBec®MED BB207-MED respirators are personal protective equipment (PPE) designed to protect against the inhalation of aerosols and dusts, in accordance with protection class FFP2, as defined in standard EN 149:2001+A1:2009. This PPE reduces the likelihood of the wearer's breathing air coming into contact with microorganisms, blood and body fluids. It can be used for infection control purposes. The mask is not intended for purposes other than those described and must not be used in these cases.

### Medical Purpose (Regulation 2017/745)

A non-sterile, form-shaped filtering mask designed to be placed over the nose and mouth of a healthcare worker to permit normal breathing while protecting the patient from large particles (e.g., body fluids, airborne particulate materials) and small particles (e.g., bacteria) during medical, surgical, dental, and isolation procedures. The risk of the direct transmission of germs is being minimised. It does not include an antimicrobial agent(s). It is made of multiple layers of non-woven polymers to produce a soft, flexible mask that will create an airtight seal against the user's face and secured using a forming nosepiece (metal wire). The device is intended for short term single use. The medical device is suitable for patient use.

### Warnings and limits of use

Respiratory protection can only be effective if the appropriate adjustments are made to ensure tightness and the mask is worn during the entire stay in the danger zone.

This product is NOT suitable for persons with beards or other facial hair in the area where the mask is worn, as proper sealing cannot be guaranteed.

Do not use the respirator in an environment where metal can cause a hazard because the nose clip contains metal.

Leave the danger zone immediately if the quality of the air you breathe is poor, if the concentration of contamination is harmful to health or life, if breathing becomes difficult, or if pain or dizziness occurs.

Only use the mask in adequately ventilated areas and not in an oxygen content  $\leq 19.5\%$  or in a potentially explosive atmosphere.

The product does not protect the user from gases or vapours.

The respirator is not suitable for protection against air pollution/concentrations that are unknown or present an immediate danger to life and health.

The product is only intended to be worn for one shift ( $\leq 8$  h).

Dispose of and replace respiratory protection immediately if it has been contaminated with blood or other infectious materials, if it has been damaged or at the end of your shift.

This product is not reusable and must not be modified or repaired.

Masks without a valve can reduce the release of exhaled microorganisms and thus the contamination of the environment.

This product cannot eliminate the risk of contagion or infection.

Failure to follow the instructions and warnings may reduce the effectiveness of respiratory protection.

### Storage

The product must be stored in a clean and dry environment within a temperature range of  $-20\text{ }^{\circ}\text{C}$  to  $+30\text{ }^{\circ}\text{C}$  and at a relative humidity of  $< 80\%$  in the original packaging.

### Instructions for fitting the mask

Make sure your hands are clean before unpacking the mask.

Do not open the foil packaging until immediately before use.

Check all mask components for damage before putting it on.

- (1) Unfold the mask and bend the nose clip until it is completely flat. Then preform the nose clip into a slight curve. Make sure the kink in the middle of the nose clip is flattened.
- (2) Now place the mask from below over your chin and nose by holding the ear loops.
- (3) Then pull the ear loops over your ears.
- (4) Now adjust the nose clip to your nose shape with both hands.
- (5) A leakage check should be carried out before entering the workplace or danger zone.

### Instructions for removing the mask

To take off the mask, pull the ear loops off from back to the front over your ears. Do NOT touch the potentially contaminated mask body. Dispose of the mask properly according to local regulations.

### Leakage check

1. Gently place both hands over the mask without letting it slip.
2. Breathe out strongly.
- 3a. If air leaks in the nose area, readjust the nose clip. Check for leakage again.
- 3b. If leakage occurs at the edge of the mask, adjust the fit of the mask and ear loops. Check for leakage again.
4. If a proper fit cannot be achieved even by readjustment, the mask will not function properly. In this case, do NOT enter the danger zone!

### Explanation of the symbols

Follow the instructions for use

End of storage period  
Format: YYYY/MM

**NR**

Not reusable

Max. temperatur range during storage

**LOT**

Batch number

Max. relative humidity during storage

**MD**

Medical device

Dispose properly  
(according to local regulations)

**UDI**

Unique device identifier

Keep away from sunlight

Caution

Do not re-use

Manufacturer

Do not use if package is damaged and consult instructions for use

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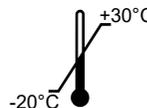
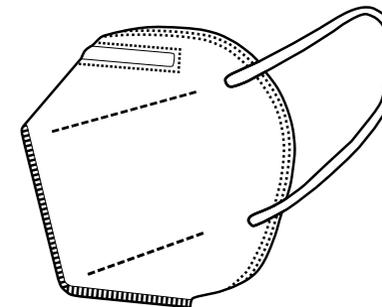
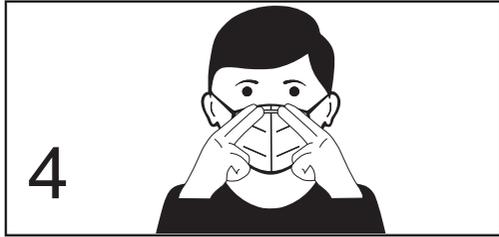
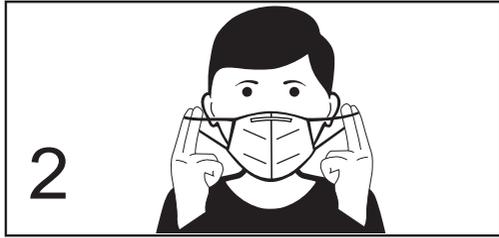
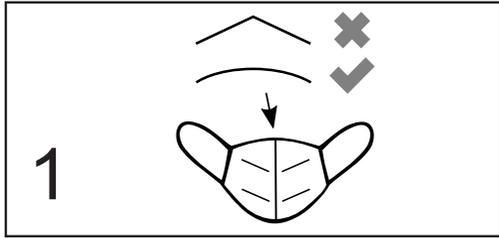
## EN Instruction for use

Medical filtering half mask  
Model BB207-MED

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### EN Reporting requirements of the user

Should any significant product-related incidents occur, the manufacturer and your relevant authority must be informed.

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### EN Certifications

These products bear the CE mark in accordance with the requirements of EU regulation 2016/425 with the notified body: 0445. They have a type approval and are checked annually by BEV - Bundesamt für Eich- und Vermessungswesen, Arltgasse 35, 1160 Wien, AT, number of the notified body: 0445.

Further these products are CE marked in accordance with the requirements of Article 20 of EU Regulation 2017/745.

The applicable legislation can be found in the CE certificate and the declaration of conformity under:  
<https://www.micurapharm.de/de/Zertifikate>

<https://www.micurapharm.de/de/Zertifikate>